DE

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Reprocess the product **prior to first use** as well as **after each use**.

General

The connectors must not be connected to other connectors when reprocessed. For safe disconnection of swivel connectors, VBM offers the uncoupling wedge (REF 60-60-999). During reprocessing connectors may remain connected in catheter mount tubings. To connect or disconnect silicone respiration tubings hold them at the cuff to prevent material tears. Please note that the products must not be in contact with hard or sharp-edged objects. Silicone materials must not be in contact with silicone oils and fats. The product lifetime of the reprocessed products depends on a careful cleaning and disinfection. Please note that each reprocessing accelerates the natural ageing of the material and reduces the lifetime of the product. Therefore, the products may not be reprocessed more than 100 times.

Warning

After each reprocessing or before each use the products need to be checked carefully for damages (cracks, tears, etc). Damaged products must not be reused!

Cleaning

For mechanical cleaning VBM recommends pH-neutral agents. Soap-suds or alkaline agents can also be used if a citric acid-based neutralisation agent is used afterwards. Abrasives, hard brushes or other materials, which might damage the surface, must not be used. Eliminate all residues of the cleaning agents by thoroughly rinsing with deionised water.

Disinfection

For disinfection use commercially available disinfectants for instruments which have been tested and approved for silicone or thermostable plastic materials. Disinfectants containing compounds similar to phenol or alkylamines (such as glucoprotamine) are not suitable and may result in hardening of the material. Strictly adhere to the immersion time and application concentration recommended by the manufacturer. Eliminate all residues of the cleaning agents by thoroughly rinsing with deionised water. For disinfection VBM recommends thermal disinfection (93°C/10min). To prevent materialdependent ageing the temperature for drying should not exceed 95°C.

For reprocessing VBM recommends the following products from ECOLAB:

Manual treatment:

Sekusept® Powder classic 2% concentration for application

Sekusept® active 2% concentration for application

Thermal-neutral treatment:

Sekumatic® FRE 0.5% regulated by machine (Miele Vario Programme)

Thermal-alkaline treatment:

Sekumatic® FR 0.5% regulated by machine (Miele Vario Programme)

Sekumatic® FNZ 0.3% neutralisation (Miele Vario Programme)

The agents mentioned above have to be used according to the instruction for use by ECOLAB (ECOLAB Germany – Applications Engineering – Phone: +49 (0) 211 9893-0 – www.ecolab.com).

Sterilisation

For sterilisation VBM recommend a validated steam sterilisation process according to DIN EN ISO 17665-1 at 132/134°C / 270/273°F (5-18min). Prior to each sterilisation process, the products must be cleaned, rinsed with water and dried. Residues of the cleaning agents and disinfectants could cause hardening or fracture of the material case of silicone. In case of PSU, residues could cause cracking or damage of the material.

Storage conditions

On the following conditions our products can be stored at least for five years:

- Temperature: 5°C to 40°C
- Humidity: 30% to 75%
- Beware of direct solar radiation and artificial light
- Do not store silicone products with PVC or rubber products

Disposal

Used or damaged products must be sterilised before disposal. Our products can be disposed according to the valid legal regulations.

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