

VBM Surgicric I


Surgical Cricothyrotomy Kit used for emergency airway access in case of upper airway obstruction when conventional endotracheal intubation can not be performed.

Components:


- Scalpel #20
- Tracheal Hook with blunt tip
- curved dilator
- cuffed tracheal tube (7.0 mm) with moveable flange
- Syringe
- Necktape
- Extension tubing




THE RAPID FOUR-STEP TECHNIQUE

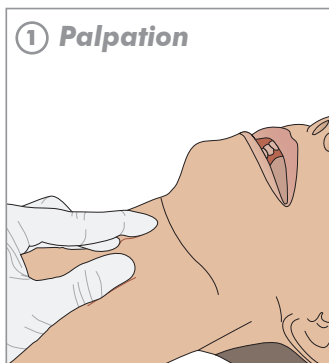


① **Palpation**
Identification of the cricothyroid membrane and stabilization

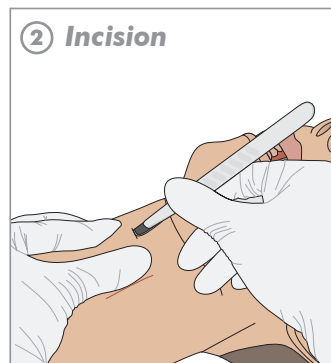

② **Incision**
Stab incision through the skin and membrane with the scalpel

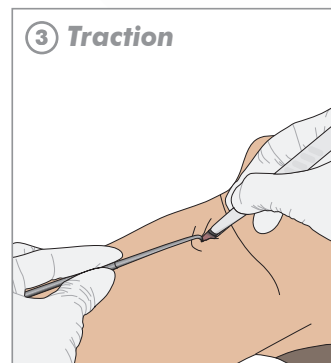

③ **Traction**
Caudal traction on the cricoid cartilage with the tracheal hook

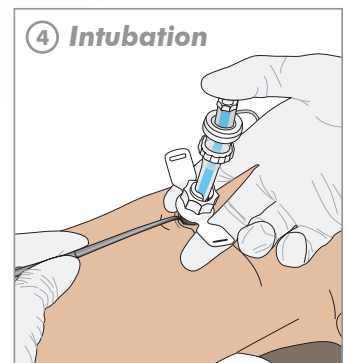

④ **Intubation**
Intubation of the trachea with the dilator and tracheal tube

ORDER INFORMATION

Surgicric I
for single use, sterile

REF 30-08-007-1 Box 1



VBM **Medizintechnik GmbH**

Einsteinstrasse 1 | 72172 Sulz a.N. | Germany

Tel.: +49 74 54 / 95 96 10 | Fax: +49 74 54 / 95 96 33 | e-mail: info@vbm-medical.de | www.vbm-medical.de

